




EndoGuard. Perfect view. Optimal safety.

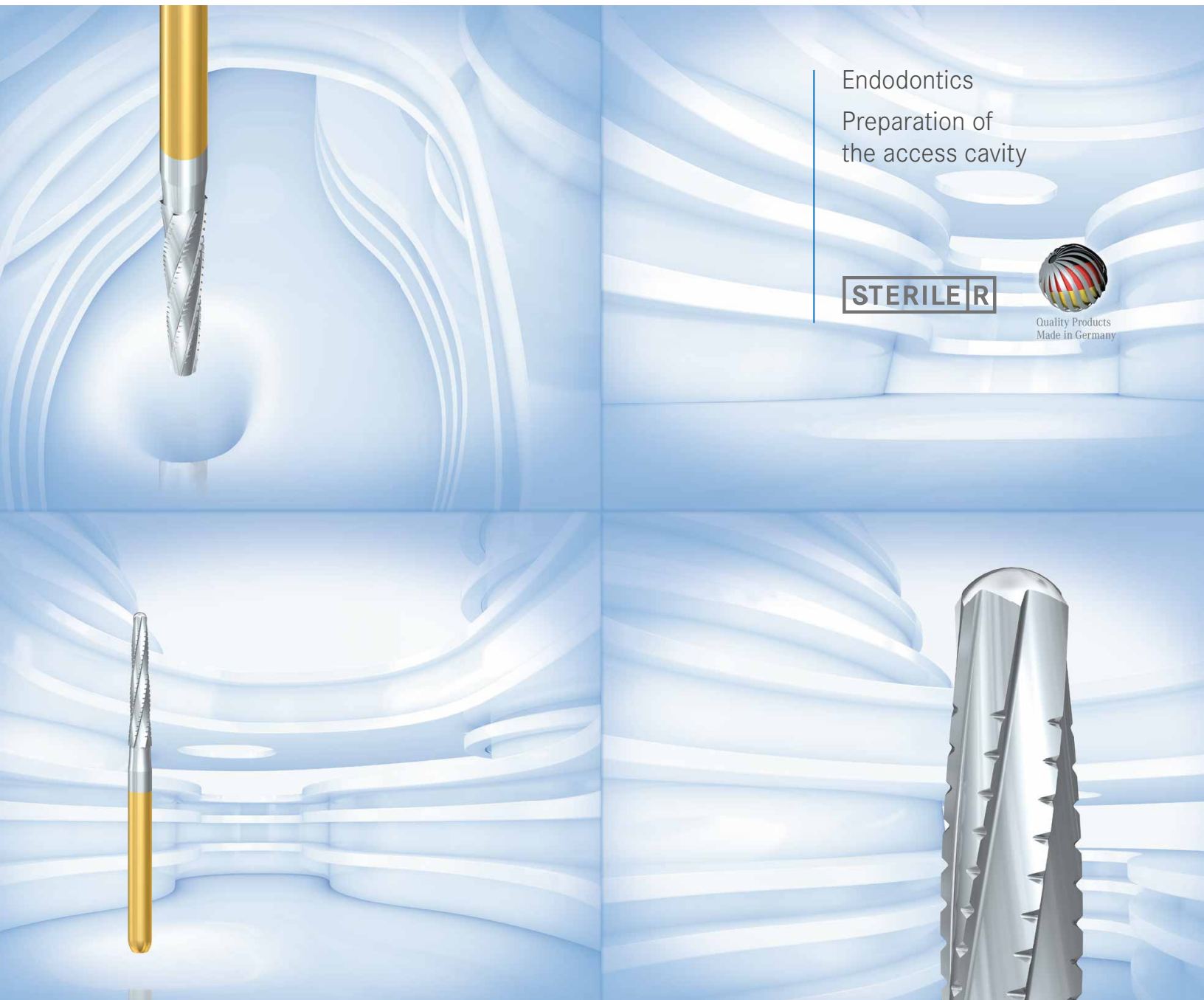

Endodontics
Preparation of
the access cavity

STERILE R



Quality Products
Made in Germany

Set the course for endo success.

The Komet® EndoGuard bur sets the course for success from the beginning of endodontic treatment, creating an ideal access cavity in minimal time and with minimal effort before subsequent preparation of the root canal.

Used immediately after the pulp chamber has been opened, the EndoGuard bur efficiently and safely simplifies the creation of a straight-line access cavity.



Perfect access cavity

Following the initial opening of the pulp chamber, dentin overhangs are removed to optimize visual examination and assessment. The proper technique provides straight-line access to the root-canal system, thereby minimizing the risks of inadvertent canal transportation and file fracture.



Protection of the pulp chamber floor

The EndoGuard features a safe, non-cutting tip to protect the floor of the pulp chamber and to prevent excessive removal of hard dental substance.




No preparation of undercuts

Featuring a tapered shape to avoid the preparation of undercuts, the EndoGuard ensures that no infected tissue is left behind in the pulp chamber.



Efficient cross-cut toothing

With a cross-cut toothing design, the EndoGuard facilitates efficient, smooth operation to maximize instrument control at all times.

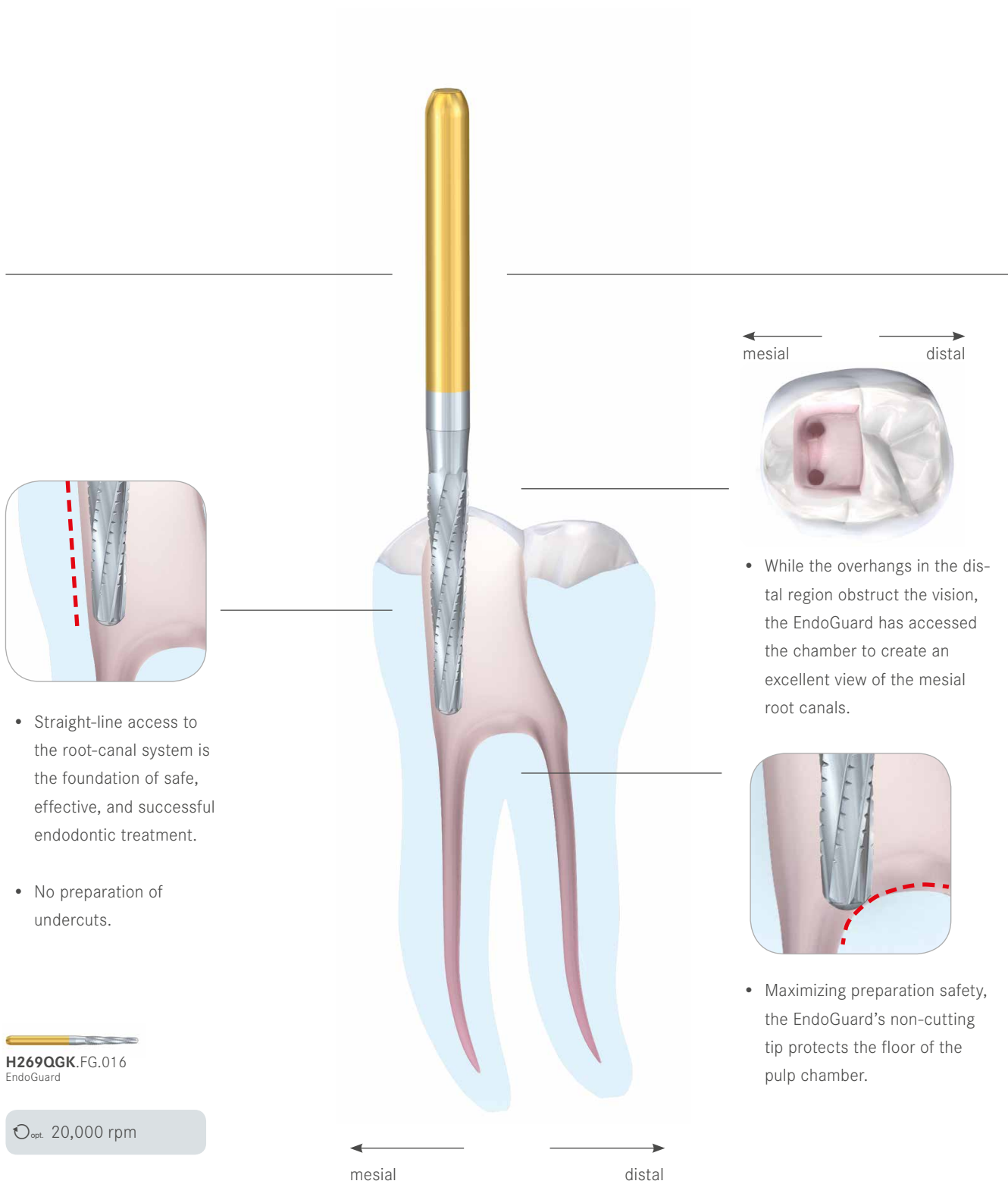

Komet USA LLC
3042 Southcross Blvd, Suite 101
Rock Hill, SC 29730

Phone 888-566-3887

Fax 800-223-7485

info@kometusa.com

www.kometusa.com

Some of the products and designations mentioned in the text are trademarked, patented or copyrighted. The absence of a special reference or the sign ® should not be interpreted as the absence of legal protection.

This publication is copyrighted. All rights, also with regard to translation, reprint, and reproduction (also in the form of extracts) are reserved. No part of this publication may be reproduced or processed using electronic systems in any form or by any means (photocopying, microfilm, or other methods) without the written permission of the editor.

Colors and products subject to change.
Printing errors excepted.

As of August 2015

